

Decontamination of water

as at: 15 January 2007

For the decontamination of water lines and the prevention of biofilm formation in dental treatment units.



- *Removes organic as well as inorganic residues e.g. limestone in a safe and reliable way*
- *Effective against a broad spectrum of germs*
- *Prevents a fast new build up of biofilm*
- *Not corrosive*
- *Ready for use*
- *No permanent adding*

Composition

ethylene diamine tetraacetate, p-hydroxy benzoic acid ester, polyhexamethylenebiguanide, phenylalanine

Sphere of activity

- *Bactericidal*
- *Fungicidal*

Clinical data

1. Institut W. Kurth, Überlingen, 06.02/ 08.02
2. Institut Heppeler, Phys. + ecolog. compatibility, 02.99

You are welcome to request these clinical data with us.

Field of application

Solution ready for use for application in the Weekend-System II/ALPRO-WTA once a week. Serves for the disinfection' and long-lasting removal of the biofilm in the water pipes of all dental treatment units (including instruments and tumbler filler).



Shelf life

2 years

Dosage

Before starting the application (weekend, holidays or over night) fill the holding tank with Bilpron up to the mark by use of the Bilpron dosage beaker. Remove handpieces from the instrument tubes. Operate the decontamination system according to the operating instructions. Fill each water line (all instrument tubes and the tumbler filler) with Bilpron until it comes out of the tubes and the tumbler filler (clearly recognizable by the blue colour). Afterwards turn off the dental unit. Allow the Bilpron to remain in the lines until the practice starts to work again.

Decontamination of water

as at: 15 January 2007

When the dental unit is put into operation again, turn off the decontamination system and flush all instrument tubes including the tumbler filler until clean water comes out. For further advice, please consult the instructions supplied with the ALPRO Weekend-System.

Application is only to be carried out by a suitably qualified member of staff.

Delivery forms

Art. no. 3180 5 ltr. can

Art. no. 3179 Bilpron StarterSet

Accessories

Art. no. 3093 Dispenser tap

Art. no. 3091 Dosage pump

Art. no. 3096 Can wrench

Art. no. 3081 Beaker (500 ml)

Certification

EC-Certificate No. G1 06 03 29056 060

Cleaning- and disinfection preparations for water-bearing lines of medical and dental treatment units. Valid until 31 December 2008.